


GLADIATOR PRO


GLADIATOR PRO FACE SHIELD USER GUIDE

1. Wash your hands with water and soap according the instructions of World Health Organization in this link:

https://www.who.int/gpsc/5may/How_To_HandWash_Poster.pdf

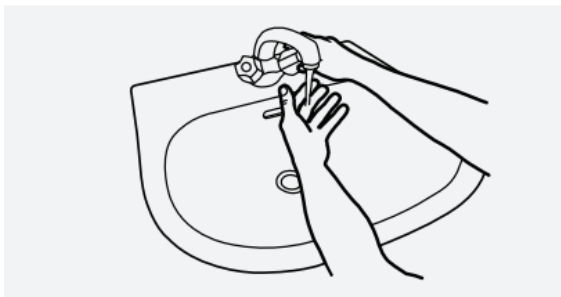


2. Please, always clean GLADIATOR PRO before using it sanitizing the Face Shield with the products recommended below.

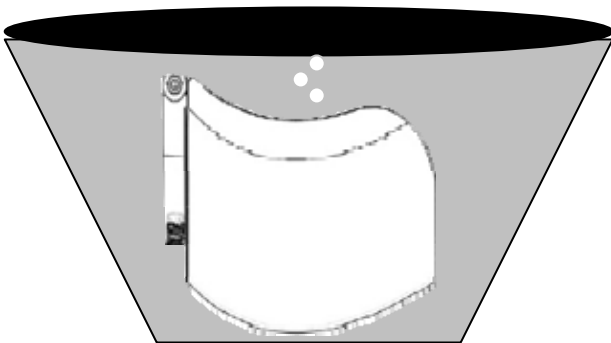

DISINFECTANTS/SANITIZERS:

- Hydrogen Peroxide 28%
- Isopropyl Alcohol
- Soap Solution
- Clorox 4%

3. Insert the two pins of the polycarbonate clear lens inside the holes of the two polypropylene head bands of GLADIATOR PRO, choosing the right hole of the head bands according the size of your head in order to wear it comfortably.

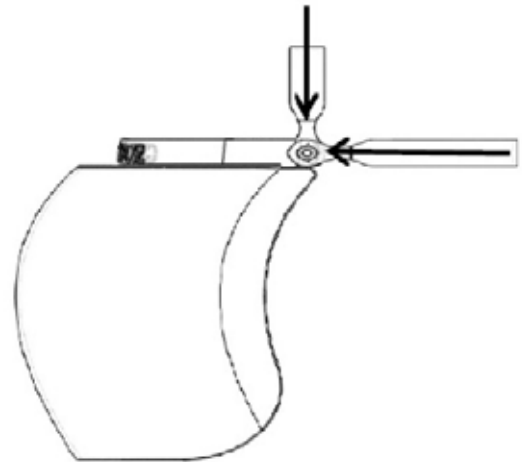

4. your gladiator pro is up & running !

